

Huddle

Clean the ACCU-CHEK Inform II Meters after every patient and prn.

- Wear gloves
- Use Super Sani-Cloth wipes only
- Turn monitor “off” and make sure there is no test strip in the port before cleaning.
- Don’t spray anything on meter or immerse in liquid
- Wring wipe before using so no fluid collects on screen
- Wipe outside of meter 3 times horizontally and 3 times vertically and around strip port
- Allow meter to air dry for 2 minutes. Do not place in unit wet.



Document cleaning by pressing 

to enter “meter cleaned” comment.

Press 

to return to the results screen, and press it again to return to the Main Menu. We are auditing who is cleaning the machines.